Contraindications/Precautions

After natural disasters and other serious trauma occur the CES Ultra™ quickly establishes the equilibrium in the central nervous system.

The induced relaxation response, resulting from use of CES Ultra™, does not in any way impair reaction time. There are no known contraindications for the use of CES Ultra™. There are, however, circumstances in which its safety has not been tested. As with the use of any device, the healthcare provider or licensed practitioner should be informed of any medications or neuro—transmitter blockers, the employment of cardiac pacemakers, any other electronic devices or insulin pumps. A CES Ultra™ unit should not be used without on—going clinical supervision in cases of severe depressive, epilepsy or pregnancy.

It is recommended that CES Ultra™ not be used while operating dangerous or complex equipment or while driving. Negative side effects are rare. Less than three percent of CES Ultra™ users report a slight headache. This is usually alleviated by simply turning down the stimulation. If the headache reoccurs during ordinary use, cease using the unit and consult your healthcare provider.

CES Ultra[™] is not a substitute for professional counseling, meditation, or constructive relationships. Used in conjunction with those efforts, CES Ultra[™] assists you in attaining a balanced emotional state.

Technical Specifications

Power Source:

9V alkaline battery

Output:

Wave Shape:

Rectangular

Amplitude:

Adjustable 0–1.5 Ma

Pulse Duration:

2 Milliseconds

(20% duty cycle)

Frequency:

100 Hz

The CES Ultra™ Kit





Copyright © 2011 Neuro Tech, Inc by permission of Neuro–Fitness LLC. If you have any questions please contact us at support@traumarecoveryinternational.com



Immediate Relief from the Effects of Trauma



The World's Premier System for quick, safe, drug-free relief of symptoms of Post Trauma Disorder – anxiety, depression & insomnia.

www.TraumaRecoveryInternational.com